



Welcome to the *Wheels Bridge*[®] presentation
of

TiVi80 – Spot Analyzer

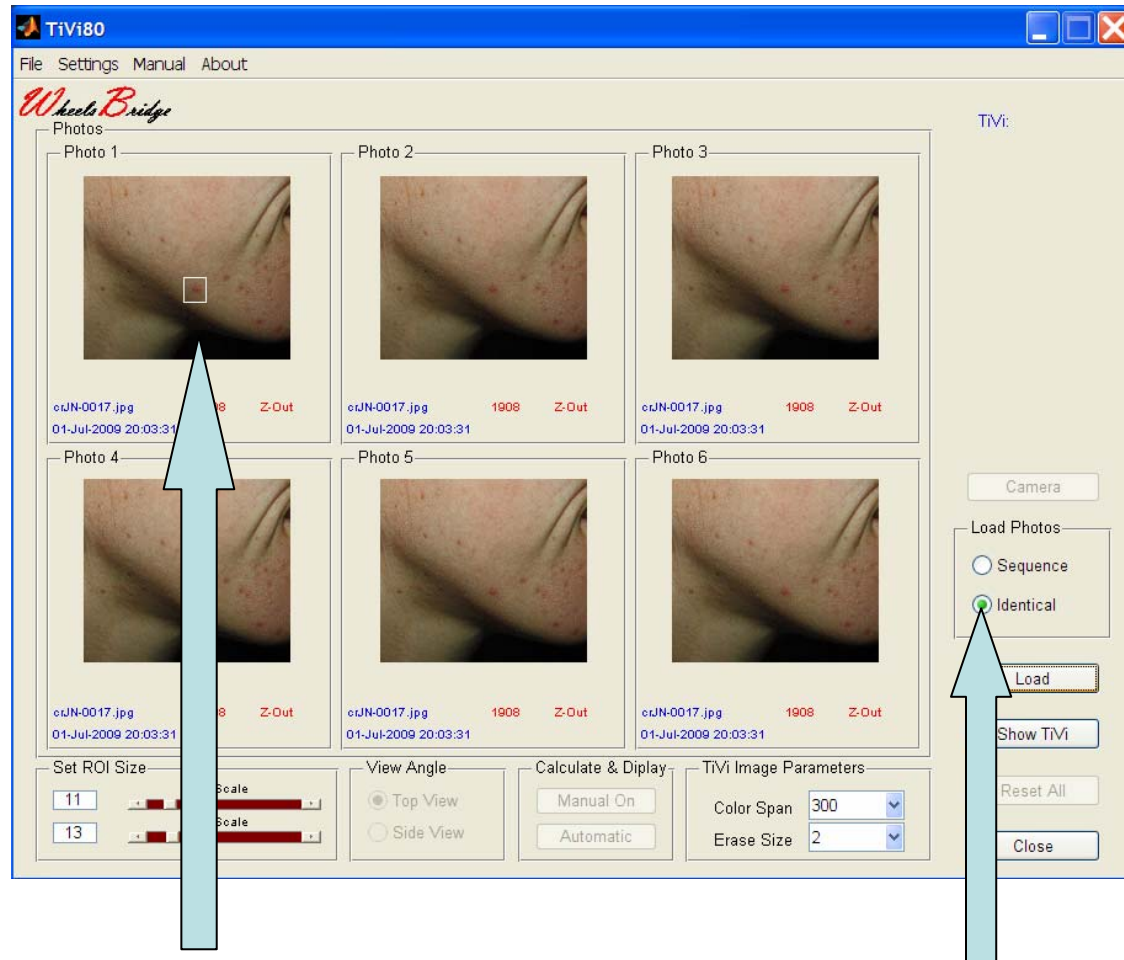


Move the pointer to a spot and click the left mouse button to automatically draw the **ROI**.

Check **Identical** and load a photo.

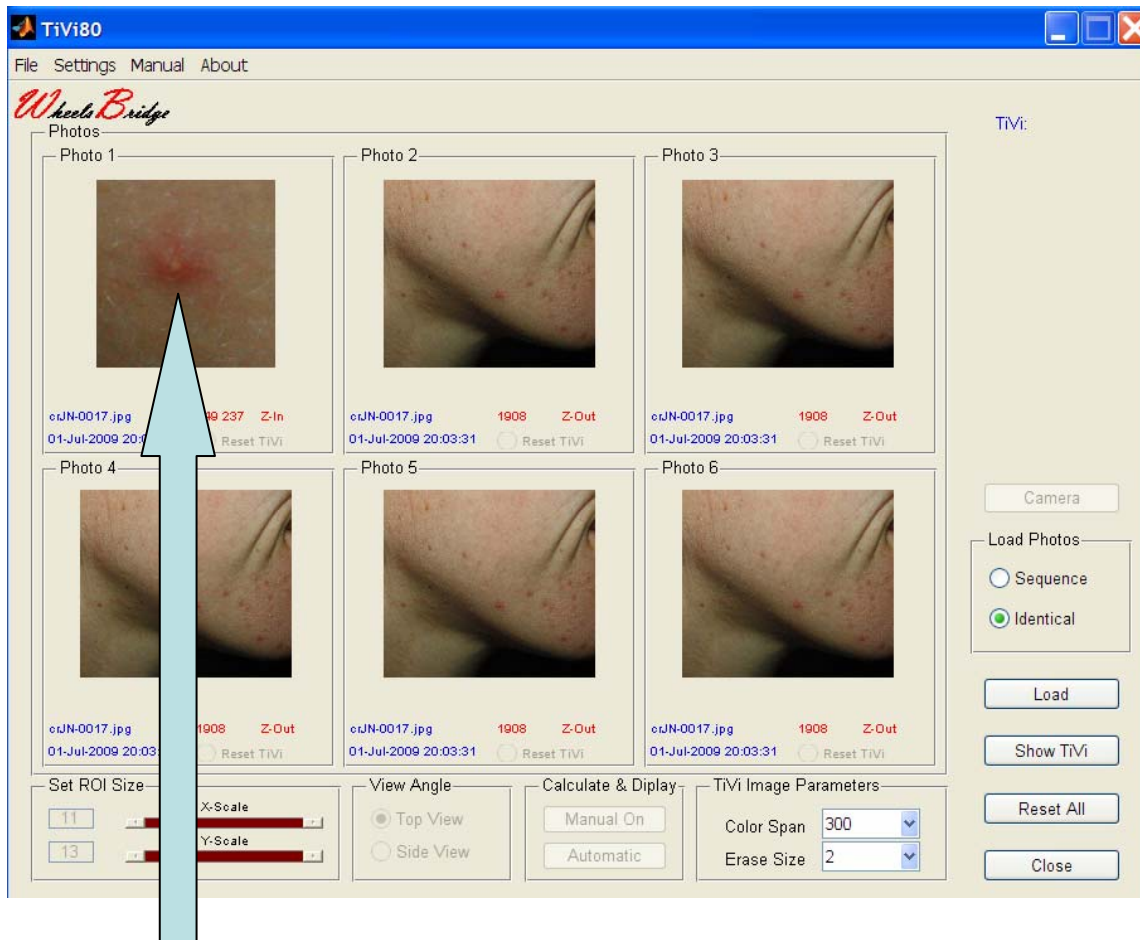


The selected spot will appear in **zoom-in** mode.

Repeat the procedure with other spots and a reference area. Click the **Show TiVi** button.

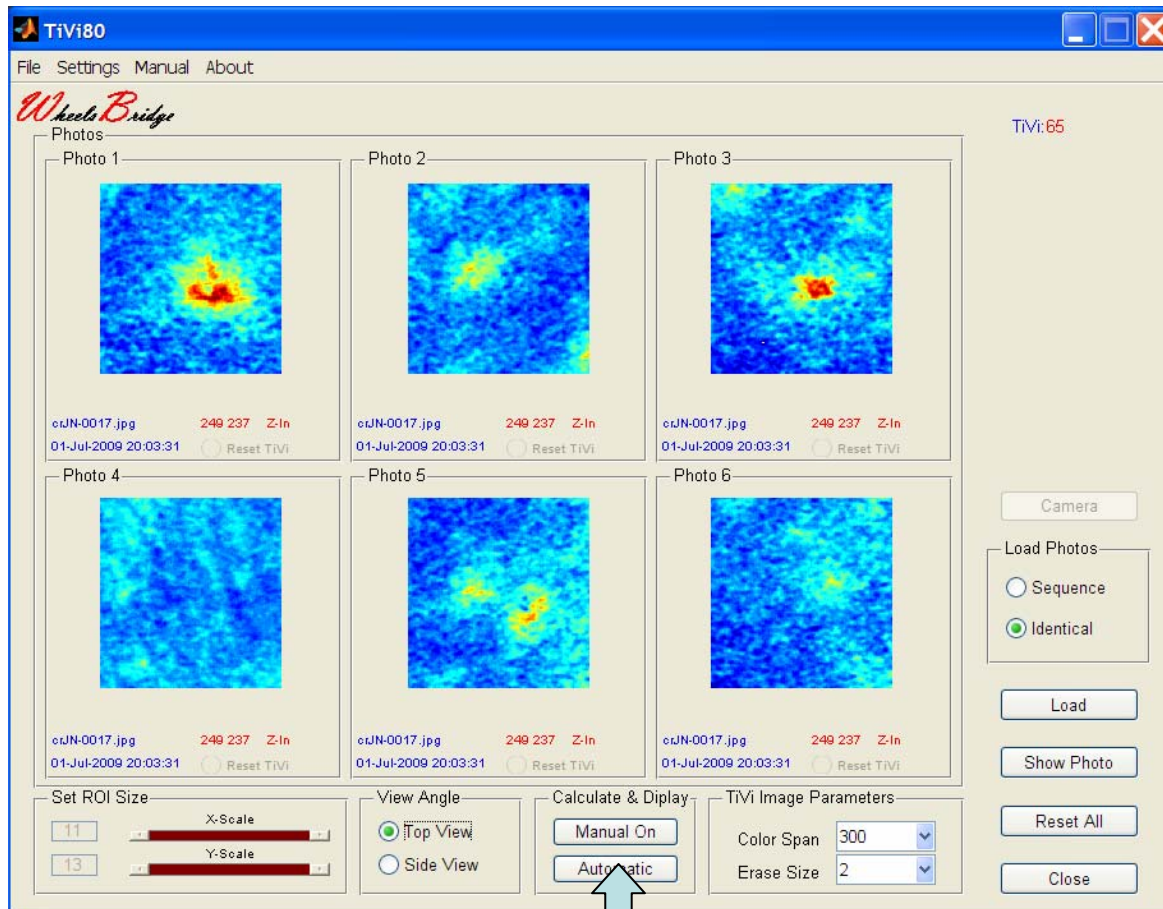


The **TiVi images** of the individual spots are displayed.


Click the **Automatic** button to display the **Erythema Chart**.

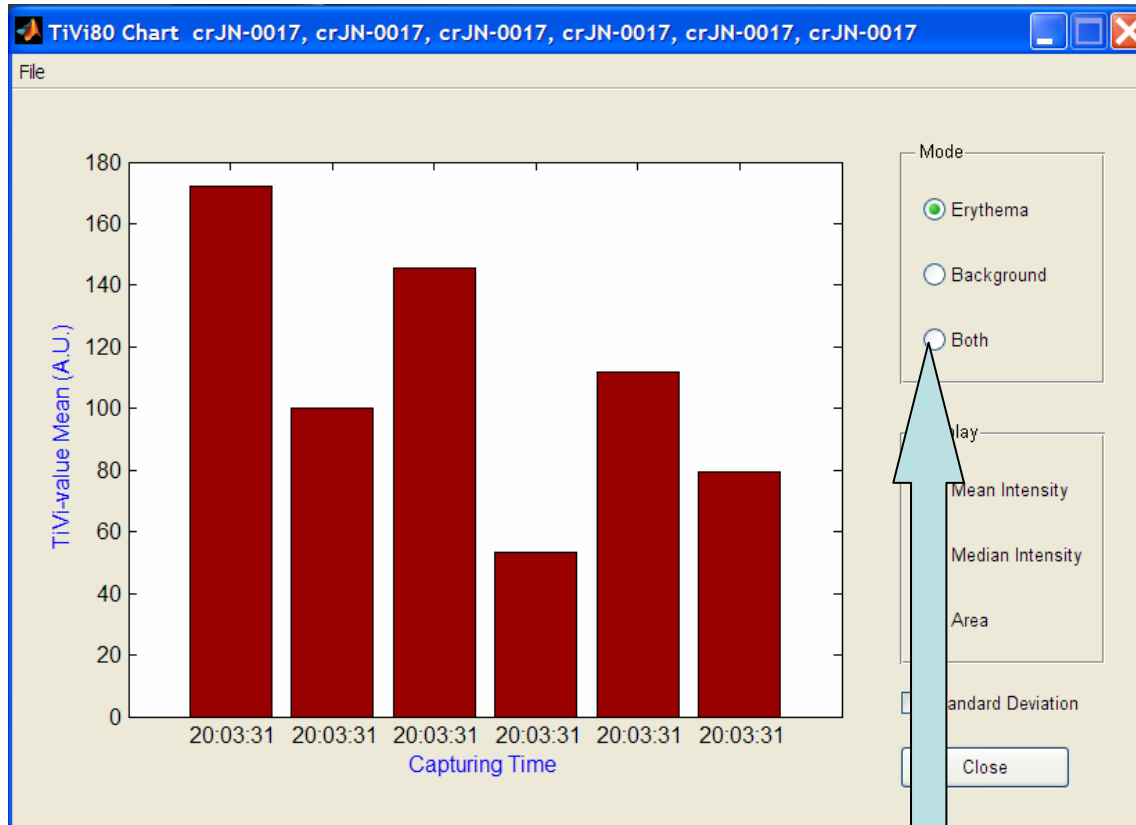

Check the **Both** button to display the **Erythema Intensity** added on top of **Background**.

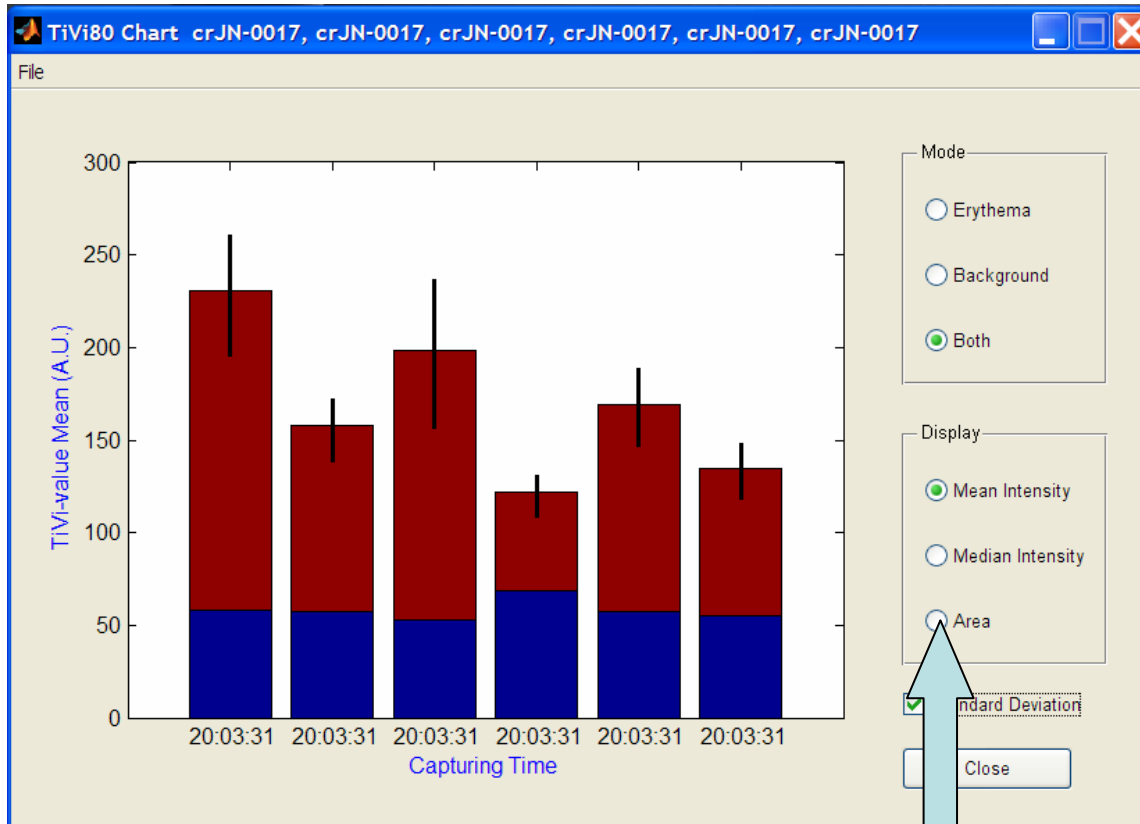

Check the **Area** button to display the **extension** of the individual spots.

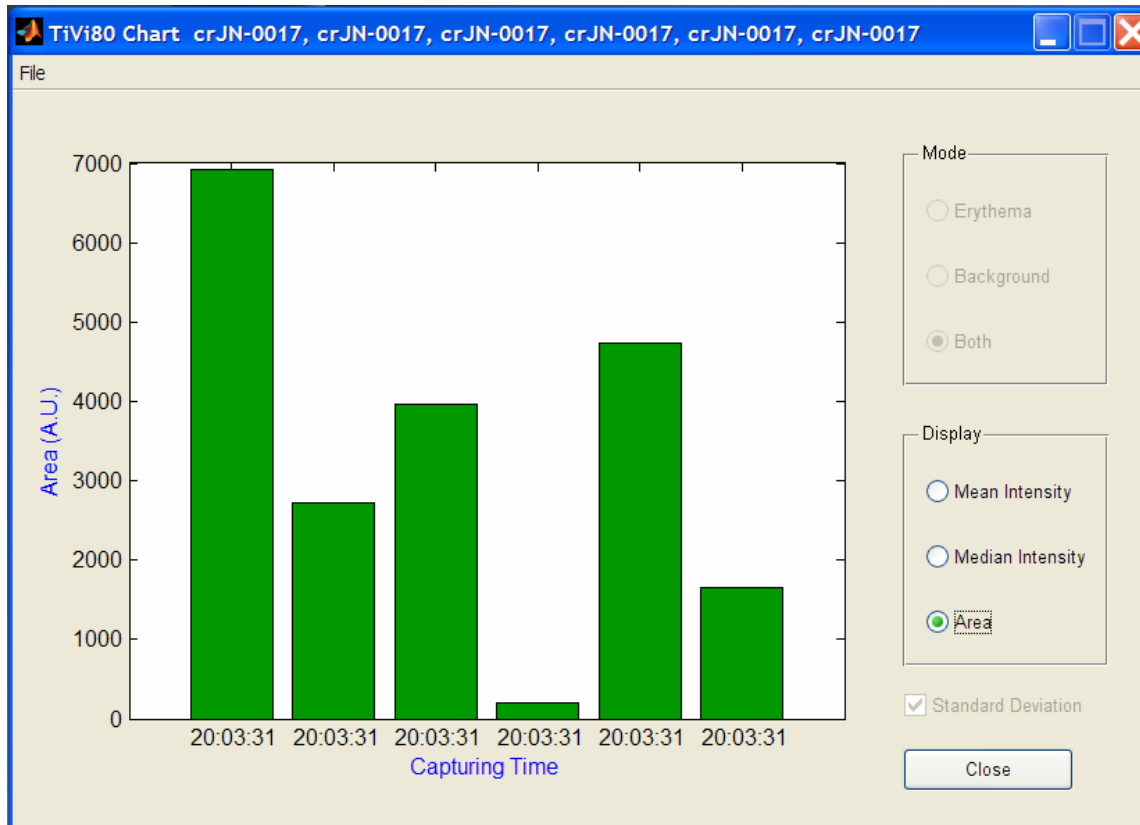

Finally add the values of this **Chart** to the **Project Diagram** to display the entire result of a test panel trial in a single window.

